




Bug Report for OsiriX




For a crash:

1. Go to the Applications/Utilities folder
2. Launch 'Console' application
3. Menu View: 'Show Log List' if not displayed
4. Find and select (see screen capture):
 - ~/Library/Logs
 - CrashReporter
 - OsiriX.crash.log
- 5.- Copy and paste the entire crash log in your bug report

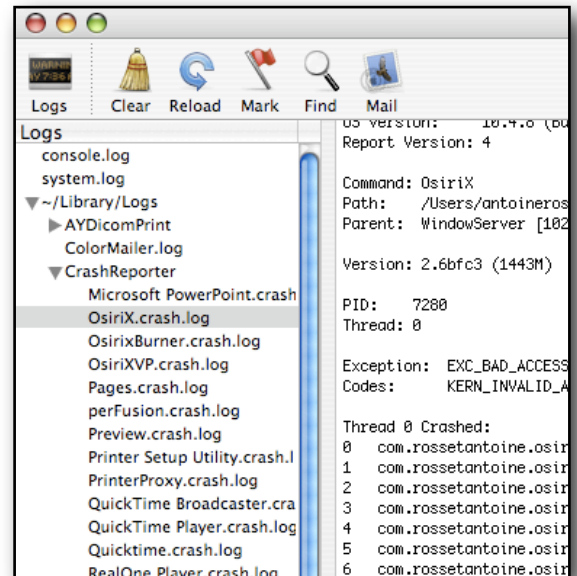

For a hang (spinning ball):

- Go to the Applications/Utilities folder
- Launch 'Activity Monitor' application
- Select OsiriX in the list (displayed in **red** for hanging applications)
- Menu View: Sample Process
- Save... button on the window top right
- Copy and paste the entire report in your bug report

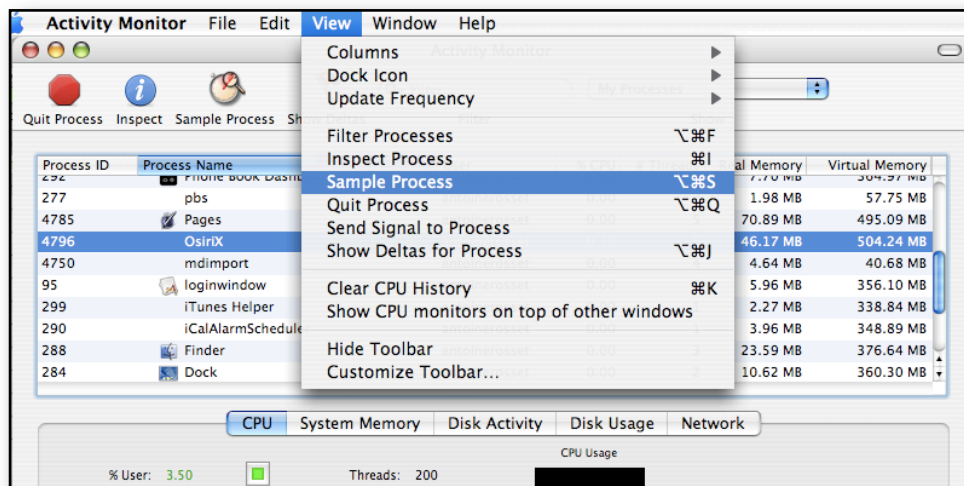

Report your bug:

Post it on Sourceforge: http://sourceforge.net/tracker/?group_id=107249&atid=647131

Or send it by email to: rossetantoine@bluewin.ch

